



**Comfort-Fit® Low Back Support** is made from neoprene, this back support offers four-way stretch for full mobility while retaining body heat to provide soothing moist heat therapy, keeping muscles loose. Fully adjustable, it's great for relieving tight or sore backs due to heavy lifting, sitting or riding for long periods of time. 30-Day Money-Back Guarantee/Six-Month warranty or as required by law.

Colors: Black, Navy & Tan.

- PC200 SMALL (AVAILABLE IN BLACK ONLY)
- PC210 MEDIUM
- PC101 LARGE
- PC103 X-LARGE (AVAILABLE IN BLACK ONLY)
- PC300 XX-LARGE (AVAILABLE IN BLACK ONLY)



## SIZING CHART

Size	Catalog Number	Length Inches/CM	Front Height Inches/CM	Pad Height Inches/CM	Pad Width Inches/CM	Waist Size Inches/CM
<b>SMALL</b>	PC200	40"/102cm	4"/10cm	8"/20cm	4"/10cm	22"-38"/56cm-97cm
<b>MEDIUM</b>	PC210	40"/102cm	6"/15cm	8"/20cm	4"/10cm	22"-38"/56cm-97cm
<b>LARGE</b>	PC101	42"/107cm	7"/18cm	8"/20cm	4"/10cm	24"-40"/61cm-102cm
<b>X-LARGE</b>	PC103	49"/125cm	8"/20cm	10"/25cm	7"/18cm	41"-50"/104cm-127cm
<b>XX-LARGE</b>	PC300	58"/147cm	9"/23cm	10"/25cm	9"/23cm	51"-60"/130cm-152cm

## APPLICATION INSTRUCTIONS



1. Grasp outer ends of support and position lumbar pad at the small of the back.
2. Stretch out both ends, wrapping longer non-hook end across your body.
3. Align the hook end and attach the hook fasteners securely.
4. Center lumbar pad at the small of the back.
5. Grasp both side-pulls simultaneously and attach hook fasteners to front of support.